## FICHA TECNICA HOSPITALARIA

### SANIT MAK L

#### Propiedades

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### APPLICATIONS

Neutral non-foaming liquid additive for medical instruments lubrication. It is also an excellent protector against oxidation and corrosion, which prolongs the useful life of the medic-instruments. The product is suitable for the sterilization of material.

### DOSAGE AND DIRECTIONS FOR USE

Manual application by immersion: dilute from 0,01 to 0,1 % (from 0,5 to 5 ml of product for every 5 litres of water).

Application by utomatic machine: from 0,01 to 0,1 % (from 1 to 10 ml per wash cycle).

### SUBSTANCES THAT CONTRIBUTE TO THE CLASSIFICATION

(1-6 EO) ethoxylated C12-14 alcohols.

### PRECAUTIONS

H318 May cause an allergic skin reaction.  
P280 Wear protective gloves/ protective clothing/ eye protection/ face protection.  
P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do.  
Continue rinsing. P310 Immediately call a POISON CENTER or doctor/ physician. Keep out of reach of children. Do not ingest. Professional use only. In case of an accident, please seek medical advice at the Toxicological Information Medical Services. Tel. +34 915 620 420.  
Made in EU.

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### DOSAGE AND DIRECTIONS FOR USE

Manual application by immersion: dilute from 0,01 to 0,1 %, (from 0,5 to 5 ml of product for every 5 litres of water).

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### PRECAUTIONS

H318 May cause an allergic skin reaction.  
P280 Wear protective gloves/ protective clothing/ eye protection/ face protection.  
P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do. Continue rinsing. P310 Immediately call a POISON CENTER or doctor/ physician. Keep out of reach of children. Do not ingest. Professional use only. In case of an accident, please seek medical advice at the Toxicological Information Medical Services. Tel. +34 915 620 420.  
Made in EU.

## S.A. PRODER

Paseo Sanllehy nº 64, 08213 Polinyà - BCN (Spain)  
Tel. +34 937 132 025 - www.saproder.com

## PRODERPHARMA S.L.

Paseo Sanllehy nº 64, 08213 Polinyà - BCN (Spain)  
Tel. +34 937 133 938 - www.proderpharma.com

XXXXX

XXXX/XXXX

(01)18436532104982(17)230320(10)57996

PELIGRO  
PERIGO  
DANGER

4X5 I