FICHA TECNICA HOSPITALARIA

SANIT ESCOPE

Propiedades

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PRECAUTIONS

H319 Causes serious eye irritation. EUH208 Contains Reaction mass of: 5-chloro-2-methyl-4-isothiazole-3-one [EC no. 247-500-7] and 2-methyl-2H-isothiazole-3-one [EC no. 220-239-6] (3:1). Subtilisine. May produce an allergic reaction. P264 Wash thoroughly after handling. P280 Wear protective gloves/protective clothing/eye protection/face protection. P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P337+P313 If eye irritation persists: Get medical advice/attention. In case of an accident, please seek medical advice at the Toxicological Information Medical Services. Tel. +34 915 620 420. Made in EU.

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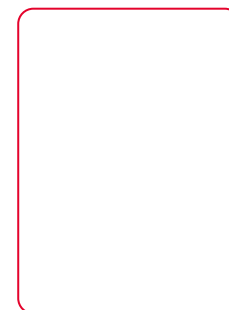
APPLICATIONS

For thorough cleaning of medical-surgical equipment by immersion, including rigid or flexible endoscopes. Suitable for ultrasonics. Biodegradable, phosphate and solvent-free. Neutral pH, compatible with all kinds of material.

DOSAGE AND DIRECTIONS FOR USE

Dilute with water to 0,1%, plunge the material for a minute and rinse. For embedded soil, increase the dose, exposure time and water temperature without exceeding 50 °C.

COMPOSITION

Enzymes: Concentrated activity of protease, amylase, lipase, cellulase and mannanase (7,5 g). 5-15% Nonionic surfactants, sequestrants, perfumes, preservatives and dyes.

PRECAUTIONS

H319 Causes serious eye irritation. EUH208 Contains Reaction mass of: 5-chloro-2-methyl-4-isothiazole-3-one [EC no. 247-500-7] and 2-methyl-2H-isothiazole-3-one [EC no. 220-239-6] (3:1), Subtilisin. May produce an allergic reaction. P264 Wash thoroughly after handling. P280 Wear protective gloves/ protective clothing/eye protection/ face protection. P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P337+P313 If eye irritation persists: Get medical advice/attention. In case of an accident, please seek medical advice at the Toxicological Information Medical Services. Tel. +34 915 620 420. Made in EU.

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S.A. PRODER